
GB **Nasal aspirator**

Instructions for use // Item no.: 79139

IMPORTANT!
THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY
AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS
SAFE!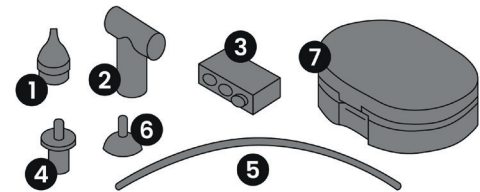

Deliverables and part description

- | | |
|--|-------------------------|
| 1. Soft silicone tip | 4. Hose connector |
| 2. Collecting container | 5. Hose |
| 3. Hygiene filter (3 pcs; can be washed out or steam sterilised) | 6. Mouthpiece |
| | 7. Hygienic storage box |

Caution!
This product is designed to support your child when they have a cold and is not a medicinal product. If your child is sick, consult a doctor.

Safety instructions

- Only use the Nasal Aspirator for its intended use
- Check the Nasal Aspirator for visible damage before each use.
- If there are any signs of damage, do not use.
- Do not use the Nasal Aspirator for nose bleeds or nostril wounds.
- Do not use on the mouth or ears.
- Not a toy, keep out of the reach of children.
- Do not leave children unattended with the product. Only for use by adults.
- Make sure that all parts are connected securely to one another before use.
- If you do not dry the parts completely before assembly, there is a risk of bacteria and mould growth.

Attention!
Never blow into the mouthpiece. Not following the instructions can endanger your child's health!

**Instruction for use
Before treatment**

If nasal secretion is too tough or dry, try to soften the blockage with an isotonic saline solution, spray or drops. In case of doubt or uncertainty, please contact your paediatrician.

Application

1. Insert a new or cleaned filter (3) into the hose connection (4)
2. Connect the hose connection (4) to the collection container (2) and the silicone tip (1).
3. Place the hose (5) onto the hose mount of the collection container
4. Place the mouthpiece (6) into the end of the hose (5).
5. Place the soft silicone tip tightly against your child's nasal opening. Caution: Place the Nasal Aspirator close to the opening of the nostril. Do not push it into the nostril!
6. Draw on the mouthpiece until the nasal discharge ends up in the collection container.
7. You can remove the hose connection and replace the filter if desired. The filters prevent the transfer of bacteria and are washable. The top is designed to avoid damaging the nasal mucous membrane.

Cleaning

1. Dismantle the Nasal Aspirator into its individual parts directly after each use
2. Wash all parts (including the filter) immediately and very carefully with warm water and mild detergent.
3. Rinse the clean parts with fresh water.
4. Let them dry out completely.
5. When not in use for long periods, we recommend that you clean the nasal aspirator again and sterilise it before the next use.
6. Clean only in the top basket of the dishwasher or as described in points 2 - 4.
7. Disinfecting only possible in the top basket of the steam steriliser or by boiling for approx. 5 minutes.
8. Do not disinfect in the microwave!
9. Let the nasal aspirator cool down and dry completely after cleaning or sterilisation.

Storage

We recommend that you store the product out of direct sunlight.

Durability and disposal

We recommend that you replace the Nasal Aspirator after a maximum of 2 years of normal use with a new one. The product can be put in with general waste and can be thermally recycled without issues.

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